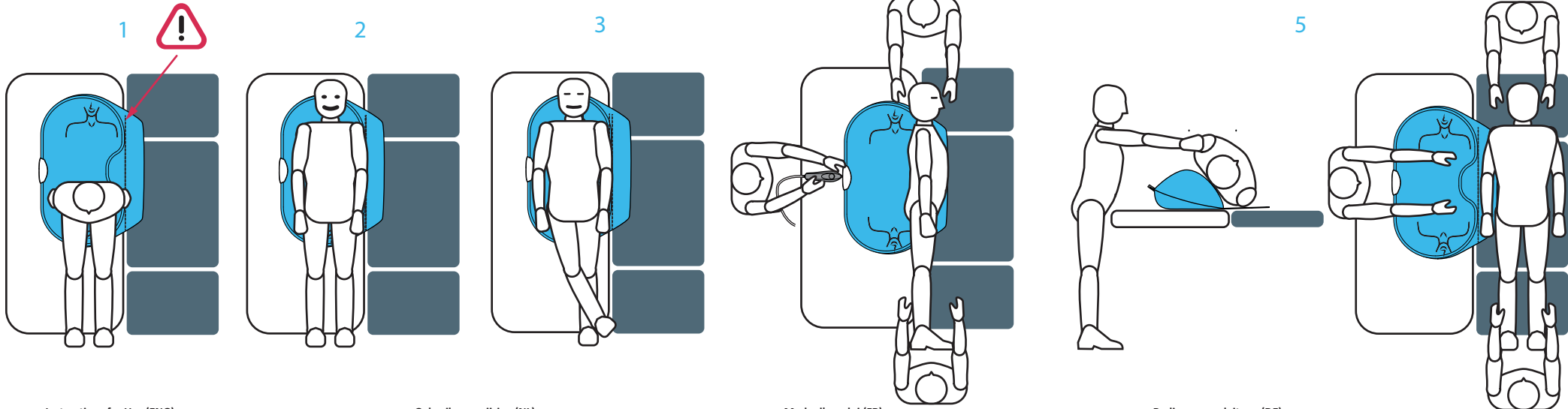
IBO

Inflatable Board

Instructions for Use (ENG) IBO (Inflatable Board)

Description

The IBO is a reusable semi-rigid inflatable board. Inflation with the Hummingbird using medical compressed air, tilts the patient from supine into lateral position.

Intended purpose

To bring a patient from supine into lateral position.
Compatible with HMB.

Operating instructions

1. Position the IBO on the bed with the valve at the tilting side. The flexible end of the IBO must be positioned onto the operating table (Fig. 1).
2. The bed where IBO and Patient are positioned on, should be positioned as close to the operating table as possible. Prevent space between both.
3. Position the patient on the IBO in supine position. Assure full trunk (thorax and abdomen) is on the IBO (Fig. 2). In case the Inflatable Prone Support is used, apply it now.
4. Cross the ankles of the patient over each other (Fig. 3).
5. One person is positioned at the head and one at the feet of the patient.
6. A third person inflates the IBO using the Hummingbird (Fig. 4).
7. Hold the valve of the IBO with one hand while pushing the HMB against it with the other hand.
8. Inflation of the IBO stops automatically when fully inflated and the overpressure valves will open. Inflating the IBO puts the patient into lateral position.
9. The patient can be rolled gently into prone position (Fig. 5).
10. Hang the IBO on a hook, it will deflate automatically within less than 10 minutes.

Warnings

The IBO has been designed for 50 uses. Leaks may occur after more than 50 inflations.
Only inflate using the Hummingbird with compressed medical air at 4 – 6 bar. Higher pressures may cause damage.
Stop inflation of the IBO when the patient is in stable lateral position or when the overpressure valves open.
Only for patients up to 150 kg.
Store under normal warehouse conditions.

Cleaning instructions

If required, the IBO can be cleaned with a soft cloth, using water/soap solution, isopropyl alcohol, chlorine dioxide or quaternary ammonium.

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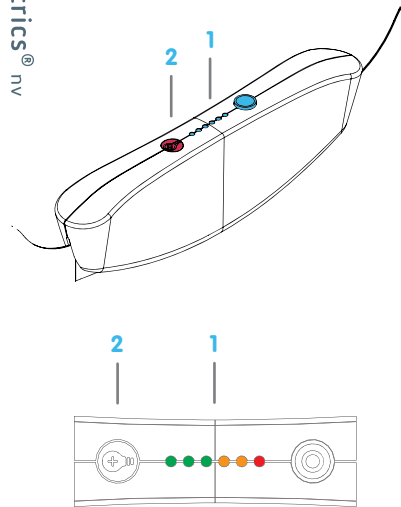
Instruction for use (ENG) (cont'd)

Parts

1. LED lights
2. Activation button
3. Battery
4. Electronics

Explanation of the LED code (1)

When using IBO or after pressing button (2), the LED lights indicate how often the IBO was used:

● ○ ○ ○ ○	1-9 times
● ● ○ ○ ○	10-19 times
● ● ● ○ ○	20-29 times
● ● ● ● ○	30-39 times
● ● ● ● ●	40-49 times
● ● ● ● ●	≥ 50 times

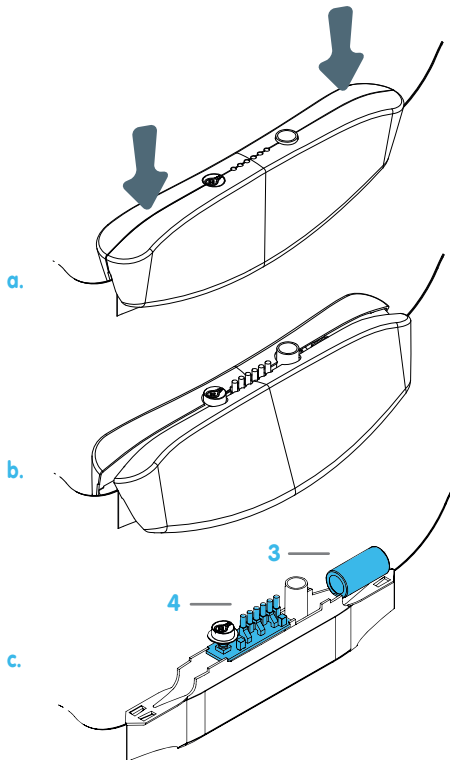
The IBO was designed for 50 applications. Leaks may occur after more than 50 inflations.

Instructions for disposal

The IBO may be thrown away with other hospital waste after the battery (3) and electronics (4) have been removed.

Remove the electronics by opening the white valve:

- a. Hold the IBO by the valve and press down on hard surface.
- b. Housing can be opened.
- c. Remove the battery (3) and electronics (4) and dispose these correctly.
- d. The IBO can now be disposed with the other hospital waste.

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