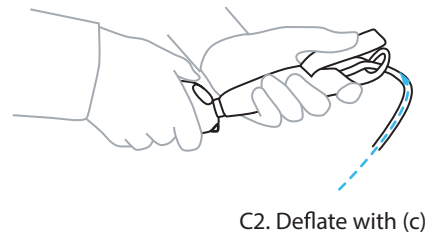
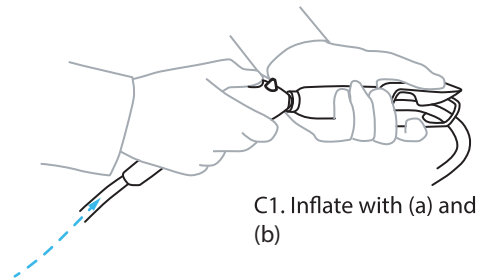
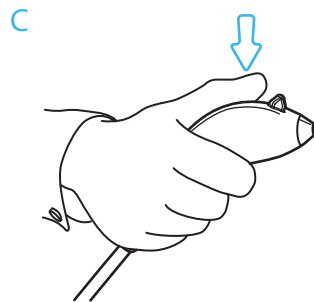
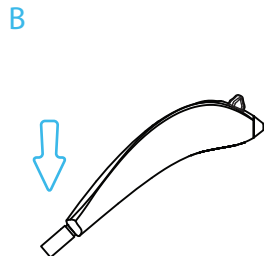
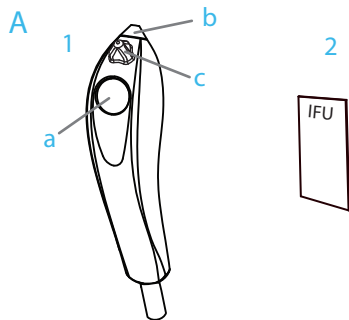
HMB

Hummingbird

 **Ergotrics®** nv

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Instructions for use HMB (ENG)

Content (A)

1. Air pistol with activator button (a), nose (b) and cone (c)
2. Instructions for Use

Installation instructions (B)

Connect the HMB to standard low-pressure hose (ID=6mm). Use hose with CE marking for compressed medical air.

Use (C)

- C1. Inflate with nose (b) and activator button (a):
 - Put the nose (b) of the Hummingbird against the valve of the Ergotrics inflatable device.
 - Push the activator button (a) to release compressed medical air. Release activator button (a) to stop the air flow.
- C2. Deflate with cone (c):
 - Push the cone (c) against the valve of the inflatable device to open the valve.
 - This will release air, when the patient (or other weight) is on the Ergotrics inflatable device. Disconnect the hose with the HMB from the compressed air connection after use.

Intended purpose

Inflation and deflation of compatible Ergotrics inflatable medical devices using medical compressed air (4-6 bar).

Warnings

Do not use with any gas other than compressed medical air at 4 - 6 bar. Prevent direct contact with skin, eyes, ears... Pressured air can cause eye and hearing damage, penetrate the skin, cause wounds and cause clots in the blood stream. Always use safety equipment, like safety goggles. Only for inflation of Ergotrics' inflatable medical devices labelled as 'compatible with Hummingbird'. Do not use in case of visible damage. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Maintenance instructions

Designed to last 5 years.
Check periodically the connection to the hose.
Clean with a soft cloth using Isopropyl alcohol or water and soap.

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